


P313--ARMU/P313--SBMU/P313--SDMU

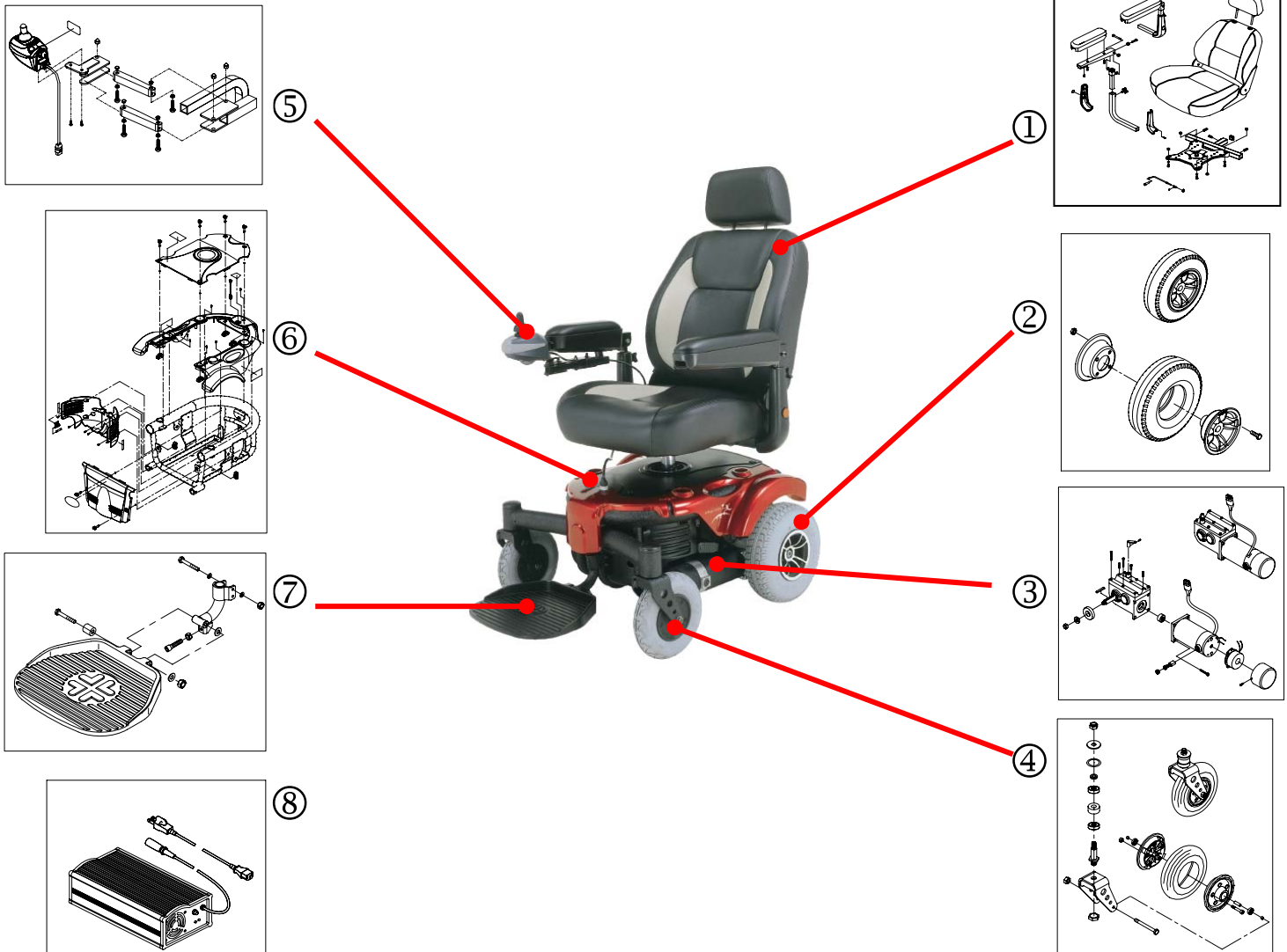

①	Seat ass'y	33100609(CASTOR ASS'Y,200x50 PU WHEEL,RH)
	32200142 (CAPTAIN SEAT ASS'Y)	⑤
	32228005 (Captain Seat Only., 18wx16dx20h, Black/Gray)	Controller
	43701072(PAD, ARMREST(LONG),BLACK)	33503058(Joystick Bracket Ass'y,Shark,Dk-Rema01)
	43701073(PAD,ARMREST(SHORT),BLACK)	64810012(JOYSTICK, DY SHARK,DK-REMA01)
②	Drive Wheel Ass'y.	64210005(CONTROLLER ONLY,DY SHARKDK-PMA05)
	32600432(DRIVER WHEEL ASS'Y.,260*85 PU)	⑥
③	Motor and gearbox complete, LH and RH.	Shroud ass'y
	62010115(MOTOR & GEARBOX,M13KA-25, LH)	30500065(UPPER SHROUD, BLACKTEXTURE, P313)
	62020115(MOTOR & GEARBOX,M13KA-25, RH)	30500066(REAR SHROUD, BLACK,P313/P314)
	62110070(MOTOR ASSY., M1KA/B, LH)	30510346(MAIN SHROUD, RED, P313)
	62120070(MOTOR ASSY., M1KA/B, RH)	⑦
	62410060(GEARBOX ASSY., M13-25,LH)	Footplate ass'y.
	62420060(GEARBOX ASSY., M13-25,RH)	33300341 (FOOTPLATE ASS'Y.W/MERITS LOGO)
④	Caster Wheel with Fork Ass'y.	⑧
	33100607(CASTOR ASS'Y,200x50 PU WHEEL,LH)	Charger unit
		65142005 (CHARGER, HP8204-5A-OFF)
		63600008(CHARGER POWERCORD W/PLUG,US SPEC)